

Contact Cleaning Brush Instructions For Use/Care

Revised 04/2008

We recommend placing this document in Central Processing for continued reference.

Device Description

The brush is designed to fit over the electrical contacts of your AEM® instrument to help remove residue left on the contact during the normal cleaning and decontamination of the instrument. These instructions are intended to be used in addition to the specific Instructions for Use supplied with each device.

NOTE: This product is supplied non-sterile and is not intended to be sterilized.

Indications/Intended Use

Use of some detergent solutions may leave a surfactant residue on the gold connector surface, depending on the strength of the solution and how thoroughly the residues are rinsed off. This may result in intermittent cord alarms. This brush is intended to remove such residue.

This device is only intended to be used on Encision's AEM Reusable and Repposable Instruments and should not be used on other instrumentation as it may damage other instruments.

PRECAUTIONS

- Treat a used instrument as a potential biohazard until cleaning and sterilization has been completed. Microscopic residues may remain after cleaning.

Instructions for Use

Before Beginning

- Thoroughly read these instructions and the instructions in the specific AEM Instrument IFU.
- Inspect the brush for wear prior to use.

Instructions


This product is for cleaning the electrical contact of Encision's AEM Reusable Instruments only.

Use this cleaning brush after following the cleaning and decontamination instructions provided with your Encision Instrument.


A. CLEANING

Step	Description
1	After cleaning and decontaminating the instrument, push the brush down onto the gold electrical contact until it bottoms out against the instrument body (as shown in the figures on this page).
2	Rotate the brush or instrument 3 to 4 complete turns.
3	Remove the brush from the instrument and wipe the gold electrical contact with an alcohol swab.
4	Sterilize the AEM Instrument in accordance with the sterilization parameters listed in the Instructions for Use/Care provided with your AEM Instrument.
5	Rinse the brush immediately following use.


ES3500 series instruments:

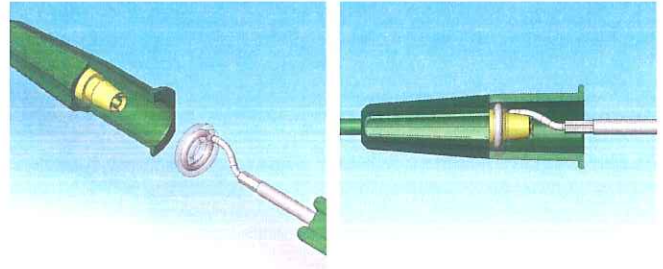


ES377x series instruments:

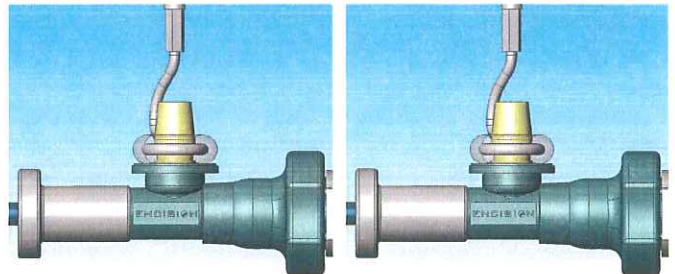

ES8000/ES8200 series Instruments:

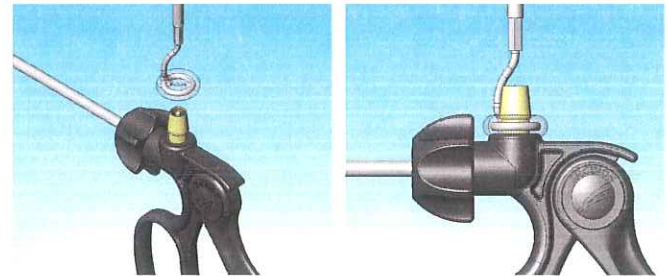

ES8000H/ES8200H series instruments:

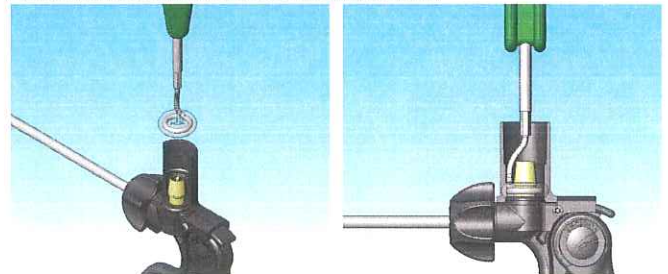

B. END OF LIFE INDICATORS

Visually inspect prior to and after use.

Discontinue use if any of the following are evident:

- Missing or broken bristles
- Shaft of brush under bristles not in circular pattern.

Cleaning Brush Instructions For Use/Care

Express Warranty

ENCISION hereby warrants to Buyer that products purchased hereunder shall be free from defects in materials and workmanship under normal use and service, as specified in this Instruction for Use/Care, for the period of one month from date of purchase, or upon reaching an end of life indicator, whichever occurs first.

Return of Used Product:

If for any reason this product must be returned to ENCISION, a returned goods authorization is required prior to shipping.

Appropriate return instructions may be obtained from ENCISION.

Manufactured By

ENCISION Inc. Boulder, CO 80301 USA

Manufactured in the USA

United States:

ENCISION Inc.

6797 Winchester Circle

Boulder, CO 80301

Ph: 303-444-2600 Fax: 303-444-2693

Ph: 800-998-0986

Symbols

REF



CATALOG NUMBER

CONSULT INSTRUCTIONS FOR USE

DATE OF MANUFACTURE

Latex Free

Products

We reserve the right to amend, modify or to change any product, to introduce new products, to withdraw products and otherwise vary product specifications at any time without notice.

Trademarks

ENCISION[®] is a trademark of ENCISION Inc.

AEM[®] is a registered trademark of ENCISION Inc.

Printed in the USA

© Copyright 2008