

AEM

2^x en·Touch[®]
SCISSORS

Quick Use Instructions

1. First Procedure and Sterilization

In the O.R. The 2x AEM[®] Scissors are delivered sterile, ready for the first use right out of the bag.

The 2x indicator is initially **black**

After the first procedure, send the scissors to SPD for sterilization.

In SPD. Sterilize the scissors after the first procedure like a reusable product. The black 2x indicator will change to **orange. Which means, there's one more use to go!**

2. Second Procedure

In the O.R. For the second procedure, the scissors will arrive with the reusable instruments.

The 2x indicator will be **orange**

After the second procedure, discard the scissors. A new scissors will be opened for the next procedure.

In SPD. If the scissors are accidentally sent to SPD with an orange 2x indicator, discard the scissors.

Orange = time for new scissors!



ENCISION[®]

AEM[®] SAFETY - PERFORMANCE - VALUE

ENCISION INC. 6797 Winchester Circle • Boulder, CO USA 80301 • T: 800.998.0986
F: 303.444.2693 • www.encision.com • info@encision.com