





*Get the Shield for Patient Safety*



## Quick Set-up

- 1** Connect the AEM® EndoShield™ (EM200) to the electrosurgical generator.
- 2** Then connect the patient return electrode to the front of the EndoShield.
-  Illuminates RED until an AEM® instrument is connected.
- 3** Connect the disposable AEM® cable to the EndoShield.
- 4** Then connect the disposable cable to an AEM® instrument.
-  Illuminates GREEN when the System is ready for surgery.

Covidien **Force Triad™**



## Low Battery

If the  or  light is flashing, it means the battery is low and the reusable AEM® EndoShield™ should be replaced.

## Troubleshooting

1. Check the setup of the AEM® EndoShield™ and Instruments as described above.

If this does not fix the issue, it may be due to the AEM® system detecting an insulation failure in the instrument. Therefore the AEM® instrument or cable may need to be replaced.

2. Replace the AEM® instrument or cable. If the problem persists, also replace the EndoShield™.

